

Molloplast® B

Permanently soft relining material for comfortable, long-term wear




- **Molloplast® B**, single component silicone comes ready to use, eliminating mixing and dosing errors.
- Easy to use with dental flasks: can be polymerized either in a water bath or using microwaves.
- Time-saving feature: can be polymerized simultaneously with acrylates.
- Bonds firmly to all types of denture acrylates, to newly made as well as pre-worn denture bases.
- Due to its specially formulated components, **Molloplast® B** stands up to the influences of the oral environment long-term.
- **Molloplast® B** is non-irritant, tissue compatible, clinically tested and scientifically documented.
- **Molloplast® B** is odour- and tasteless.
- Dentures relined with **Molloplast® B** stay comfortable for years of wearing and fully meet the demands of modern dental prosthetics.

clinically tested &
scientifically documented

Molloplast® B

Single-component silicone, homogeneous consistency – heat curing, permanently soft

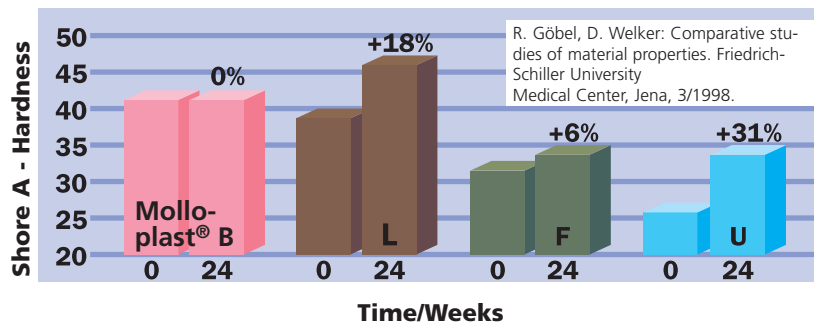

■ **Molloplast® B** contains no plasticizers and therefore will not change its consistency secondary to leaching.

■ Dentures relined with **Molloplast® B** stay soft and elastic for years.



■ **Molloplast® B** is suited for an especially large range of uses and can be used for relining, for covering of defect areas in the mouth, and for rebasing. **Molloplast® B** is also excellent for mouth guards, night-grinding guards and for modelling individual palate contours.

■ Dentures relined with **Molloplast® B** stay comfortable for years of wearing and fully meet the demands of modern dental prosthetics.



■ **Molloplast® B** has a dough-like, homogeneous consistency and comes ready to use. It can be polymerized simultaneously with acrylate when making new dentures, but can also be used for relining of already-worn dentures.

■ Once polymerized, **Molloplast® B** provides a highly elastic prosthetic base with a rubbery consistency that bonds absolutely firmly to the acrylate.



■ **Molloplast® B** is wetted completely saliva. Due to its high elasticity, it moulds intimately to the gingival contours during chewing; it thus constantly maintains the capillary film of saliva at the denture base that is so crucial for proper denture adhesion.

■ **Molloplast® B** contains no substances capable of swelling and therefore will not change shape by swelling.

■ Due to its specially formulated components, **Molloplast® B** stands up to the influences of the oral environment and resists bacteria and saliva.

■ **Molloplast® Remover** for fast and secure dissolution of the bonding between elastic relining materials and the denture acrylic. Simple application, without any time-consuming removing by grinding. No residues, no interaction during repeated relinings.



Indications for use:

Elastic cover of sharp-ridged alveolar processes with a thin tegument.

Cushioning of dentures against the mylohyoid line, mental foramen or other prominences of the alveolar ridge such as the retromolar triangle or tubera.

Lower jaws with a "tram-track"-profile.

Cushioning against the median suture.

Post-damming of the A line and to increase upper and lower denture stability by improved adhesion.

Construction of bracketless dentures and for covering congenital or postoperative defect areas with solid or inflatable obturators.

Processing technique:

Molloplast® B is simple to use. It comes ready to use and has a dough-like consistency. It can be extruded directly onto hard acrylic denture parts that have been pre-pressed with a spacer.

Once polymerized according to the instructions, **Molloplast® B** is firmly bonded to the denture acrylic. Pre-worn dentures can also be relined at any time with **Molloplast® B**.

Ordering information:

Standard packing 03000
45 g Molloplast® B

Laboratory packing 03001
170 g Molloplast® B

Combination packing 03002
45 g Molloplast® B
15 ml Primo adhesive liquid

Primo adhesive liquid 03004
15 ml bottle

LUSTROL gloss varnish
Combination packing 03008
6 ml LUSTROL gloss varnish
6 ml catalyst

Molloplast®-grinding tools
1 set (3 pcs.) 03101
1 set (2 pcs.) 03120

Molloplast®-pre-polishers 03153
1 set (10 pcs. + 1 mandrell)

10 grinding sleeves 02615

1 grinding sleeve holder 02616