making work easy

- DE
- EN
- FR
- IT
- ES
- PT
- TR
- RU
- PL
- DA
- ZH
- JA
- KO
- AR
- UK

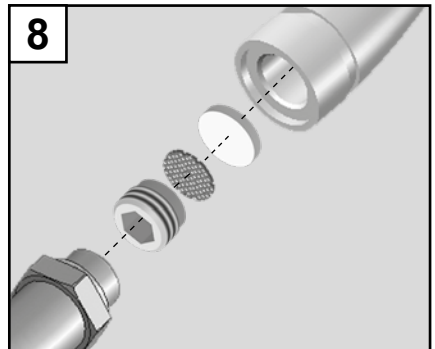
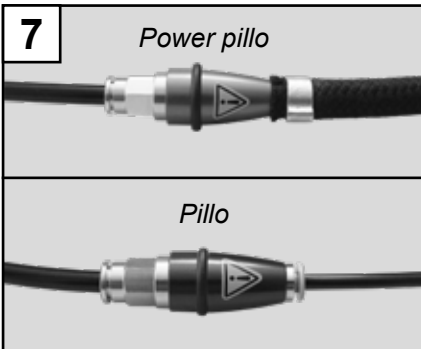
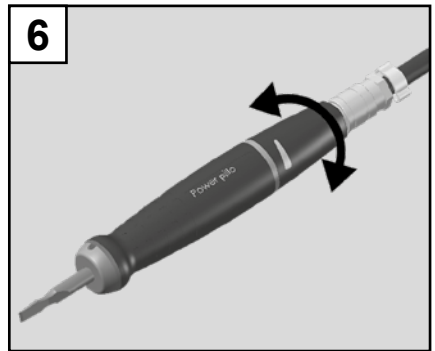
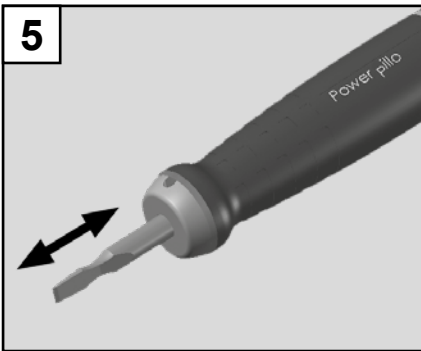
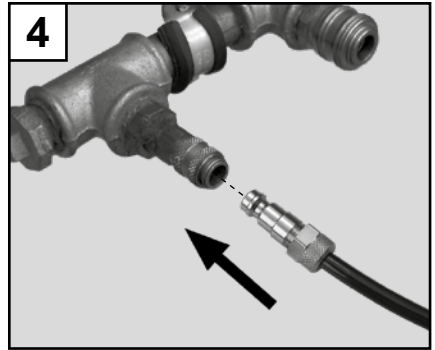
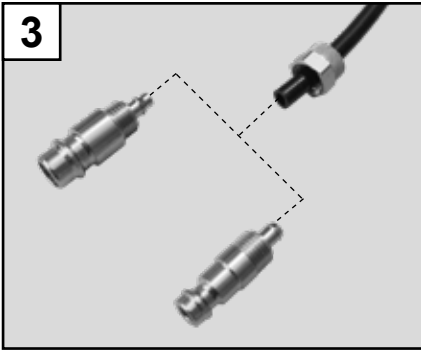
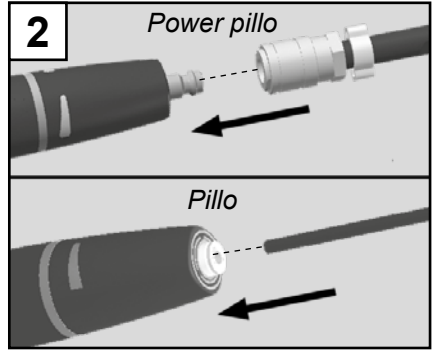
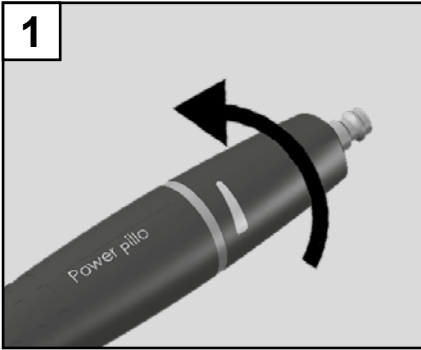
# Pillo / Power pillo

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Made in Germany

21-6558 01102019

# Pillo / Power pillo

Nr. 5022-4000 / 5022-5000

DEUTSCH

DE

## 1. Einleitung

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EN

## 1. Introduction

We are pleased with your decision to purchase a *Renfert deflasking chisel*.

*Renfert deflasking chisels* are maintenance-free, durable devices with long service lives.

**Please read the following operating instructions carefully and observe all the information they contain in order to ensure long and trouble-free service.**

**These operating instructions must always be available at the workplace.**

## 2. Application Area

The *Renfert deflasking chisel* is a pneumatically powered chisel for deflasking acrylic dentures or cast objects in dental technology operations.

**Important:**

***Pressed ceramics invested in muffles or metal framework over-pressed by ceramics must never be deflasked by a deflasking chisel as they could be damaged by the agitation.***

Compressed air is employed to oscillate the rotary piston. The impact of the rotary piston on the installed chisel transfers energy which is used to deflask.

Only compressed air may be employed to power the unit.

## 3. Hazard Information

### 3.1 Symbology

The following symbols are employed in these instructions and on the unit itself:

**Danger!**

This indicates a direct risk of injury.

**Attention!**

Always wear eye protection.

**Attention!**

Always wear a dust mask.

**Attention!**

Always wear hearing protection.

**Attention!**

Failure to observe the associated information can result in damage to the unit.

**Note!**

This provides the operator with useful information to make working with the unit easier.

Observe the operating instructions.

### 3.2 Hazard Information

- Working with the *Renfert deflasking chisel* can result in the generation of health-hazardous dusts. Always wear appropriate personal protection and ensure the work area is adequately ventilated / provided with extractor unit.
- Always wear eye protection! Working with brittle materials can generate splinters.
- Always wear hearing protection! During some tasks, the sound level can exceed 85 dB(A).
- Depending on the work piece involved, the operator's work behavior (e.g., application pressure), the setup location and - in particular - the duration of operation, the use of this device can result in hearing damage.  
The operator is obligated to have the workplace related assessment level checked in accordance with local requirements and to institute appropriate hearing protection measures.
- Always work away from yourself! Ground chisels are very sharp. There is a significant risk of injury should the chisel slip off the piece being worked.

- Use only compressed air to power the unit.
- Always disconnect the unit from the compressed air supply before exchanging or replacing chisel inserts! There is a significant risk of injury should the *Renfert deflasking chisel* start unexpectedly.
- Always disconnect the unit from the compressed air supply before beginning any maintenance tasks!
- Regularly examine the connecting hoses for damage (e.g., kinks, cracks, porosity).
- Never operate a unit with damaged connecting hoses or other defects!
- Health risk due to vibrations (hand-arm-vibrations).
- It is the responsibility of the operator that national regulations during operation and regarding a repeated safety inspection of electrical equipment are complied with. For Germany these are the regulation 3 by DGUV (German Statutory Accident Insurance) in relation with VDE 0701-0702 (Association for Electrical & Electronic Technology).
- Only spare parts and accessories supplied or authorized by Renfert GmbH may be used with this product. If other spare parts or accessories are used, this could have a detrimental effect on the safety of the device, increase the risk of serious injury and lead to damage to the environment or the device itself.
- Information on REACH and SVHC is available on our website [www.renfert.com](http://www.renfert.com), in the Support area.

### 3.3 Liability Exclusion

*Renfert GmbH* shall be absolved from all claims for damages or warranty if:

- The product is employed for any purposes other than those cited in the operating instructions;
- The product is altered in any way other than those alterations described in the operating instructions;
- The product is repaired by other than an authorized facility or if any but Renfert OEM parts are employed;

- The product continues to be employed, despite obvious safety faults or damage;
- The product is subjected to mechanical impacts or is dropped.

## 4. Commissioning

We recommend the installation of an upstream filter pressure regulator.

**Note the connection pressure! The pressure available from a compressed air network or set at a maintenance unit may not exceed the maximum level specified in the technical specifications.**

**Use only compressed air to power the *Renfert deflasking chisel*.**

- Switch the *Renfert deflasking chisel* off at the ball valve (turn the valve clockwise to its stop, Figure 1).
- Attach the connecting hose to the *Renfert deflasking chisel* (Figure 2).
- In order to access the compressed air network, take the compatible tube coupler from the enclosed set and place on the end of the tube. (Figure 3).

In addition:

- Unscrew the coupling ring from the connecting plug.
- Push the coupling ring over the connection tube.
- Connect the connecting tube with the connection plug and secure with the coupling ring.
- Attach the connecting hose to the compressed air supply (Figure 4).

**The filter housing (Figure 7) must not be removed from the connecting tube. These will help maintain the filter and will protect your *Renfert deflasking chisel* against dirt and other contaminants.**

**Never use an oil atomiser! The *Renfert deflasking chisel* is maintenance-free. An oil atomiser will produce resinous deposits in the motor or block the filter!**

## 5. Operation

### 5.1 Chisel Installation / Removal

(see Figure 5)

**Always disconnect the Renfert deflasking chisel from the compressed air supply before chisel exchange or replacement. Unplug the connector from the compressed air supply.**

**Risk of injury!**  
The employed chisels are sharp.

- To replace an insert, pull the chisel forwards and out of the tool retainer.
- Insert the new chisel into the tool retainer. The chisel must clearly lock into place in the retainer

### 5.2 Switching the Unit On and Off

The Renfert deflasking chisel is switched on and off at the ball valve (Figure 6).

- Switching the Unit On  
Turn the ball valve anticlockwise until the Renfert deflasking chisel starts.
- Switching the Unit Off  
Turn the ball valve clockwise to its stop.

### 5.3 Adjust Chisel Force

The chisel force is adjusted by turning of the ball valve (Figure 6).

### 5.4 Breaks / Conclusion of Work

- During short breaks:  
Turn off at the ball valve.
- For longer breaks or at the conclusion of work:  
Disconnect the unit from the compressed air supply.

**Pillo only:**  
**Do not loosen the connecting hose on the Renfert deflasking chisel!**

### 5.5 Operational Notes

When working with the Renfert deflasking chisel, please note the following safety information.

**Always work away from yourself.**

**Always wear eye protection! Working with brittle materials can generate splinters.**

**Always wear a dust mask! Dust from investment materials can be hazardous to your health. Always refer to the MSDS (Material Safety Data Sheets) provided with the investment material.**

**Use a dust extraction unit!**  
**The use of a dust extraction unit (refer to the accessories list) can significantly reduce health hazards.**

**Always wear hearing protection. The unit's sound level typically lies in the range of 78 dB(A). During some tasks, the sound level can exceed 85 dB(A).**

**Risk of injury!**  
**Compressed air exiting from the front of the unit represents an injury hazard. Never direct the Renfert deflasking chisel towards your face or towards other individuals.**

When working with the Renfert deflasking chisel please also ensure that the resulting splinters, dust or noise do not put others at risk.

**In order to avoid health risks (e.g., white finger syndrome) resulting from the effects of vibrations on the hands and arms, please observe the maximum exposure times specified by the relevant regulatory agency.**

## 6. Cleaning / Maintenance

**Always disconnect the unit from the compressed air supply before beginning any maintenance tasks.**

The *Renfert deflasking chisel's* motor is maintenance-free. Never use oil or grease on the *Renfert deflasking chisel*. Occasionally clean the exterior of the *Renfert deflasking chisel* with a soft, dry, lint-free cloth.

**Never use cleansers containing solvents!**

### 6.1 Cleaning the Filter

Regularly (at least every month) examine the filter in the connector fitting (Figure 7) and clean it as required.

**Use only Renfert filters!**

Filter replacement (Figure 7):

- Roll away the O-ring seal.
- Unscrew filter housing (hold up at the filter housing with the screwdriver attached).
- Using an Allan key (SW 5 mm) unscrew the sleeve.
- Remove the sieve together with the filter unit.
- Install a new piece of non-woven material and sieve making sure to adhere to the correct sequence (Figure 7).
- Screw the sleeve back in and hand-tighten.
- Screw filter housing together.
- Replace the O-ring seal

**The filter housing (Figure 7) must not be removed from the connecting tube. These will help maintain the filter and will protect your Renfert deflasking chisel against dirt and other contaminants.**

### 6.2 Chisel Grinding

Dull chisels can be sharpened by grinding them. Only tools and equipment intended specifically for sharpening may be employed.

**We can assume no liability for damages resulting from improperly ground chisels.**

## 7. Spare Parts

You can find components subject to wear and the spare parts on the spare part list in the internet at [www.renfert.com/p918](http://www.renfert.com/p918).

The components excluded from the warranty (such as consumables or parts subject to wear and tear) are marked on the spare part list.

Serial number and date of manufacturing are shown on the type plate of the unit.

## 8. Warranty

Provided the unit is properly used and with the exception of components subject to normal wear, Renfert **warrants** the all components of the *Renfert deflasking chisel* for a **period of 3 years**.

Parts which are subject to natural wear and tear (wear parts) and consumables are excluded from the guarantee. These parts are marked in the spare part list.

Components subject to natural wear are excluded from this warranty. The warranty is voided in case of improper use; failure to observe the operating, cleaning, maintenance, and connection instructions; in case of independent repairs or repairs by unauthorized personnel; if spare parts from other manufacturers are employed, or in case of unusual influences or influences not in compliance with the utilization instructions.

Warranty service shall not extend the original warranty.

## 9. Technical data

### 9.1 *Pillo*

|                             |                                      |
|-----------------------------|--------------------------------------|
| Nominal operating pressure: | 5 bar [72 psi]                       |
| Operating pressure:         | 4 – 6 bar [58 – 87 psi]              |
| Air consumption, max.:      | 9 l/min [0.32 cfm]                   |
| Piston frequency, approx.:  | 110 Hz                               |
| Ahv*):                      | 7,3 m/s <sup>2</sup>                 |
| LpA**):                     | 79 dB(A)                             |
| Dimensions:                 | Ø 29 x 157 mm<br>[Ø 1.14 x 6.2 inch] |
| Weight, approx.:            | 300 g [10.6 oz]                      |

\*) hand / arm vibration in accord. with EN ISO 5349-1

\*\*) sound pressure level in accord. with  
DIN EN ISO 11202

### 9.2 *Power pillo*

|                             |                                      |
|-----------------------------|--------------------------------------|
| Nominal operating pressure: | 5 bar [72 psi]                       |
| Operating pressure:         | 4 – 6 bar [58 – 87 psi]              |
| Air consumption, max.:      | 10 l/min [0.35 cfm]                  |
| Piston frequency, approx.:  | 130 Hz                               |
| Ahv*):                      | 5,2 m/s <sup>2</sup>                 |
| LpA**):                     | 82 dB(A)                             |
| Dimensions:                 | Ø 29 x 168 mm<br>[Ø 1.14 x 6.6 inch] |
| Weight, approx.:            | 398 g [14.0 oz]                      |

\*) hand/arm vibration in accord. with EN ISO 5349-1

\*\*) sound pressure level in accord. with  
DIN EN SO 11202

## 10. Standard Delivery

- 1 *Renfert Deflasking Chisel*
- 1 Chisel insert Nr. 5022-0200
- 1 Connecting hose
- 1 Connector fitting set
- 1 Operating instructions

## 11. Delivery Versions

- Nr. 5022-4000 *Pillo*  
Nr. 5022-5000 *Power pillo*

## 12. Accessories

- Nr. 5022-0100 Flat chisel, narrow  
Nr. 5022-0200 Flat chisel, wide  
Nr. 5022-0300 Hollow-ground chisel  
Nr. 5022-0400 Pointed chisel  
Nr. 2626-xxxx *Dustex master plus*

## 13. Troubleshooting

EN

| Error  | Possible cause   | Corrective action  |
|--|--|--|
| <b>Renfert deflasking chisel fails to start; no sound of compressed air.</b>     | <ul style="list-style-type: none"> <li>• Compressed air supply not established.</li> <li>• Ball valve on the Renfert deflasking chisel not properly / fully open.</li> <li>• Kink in connecting hose.</li> <li>• No compressed air.</li> </ul> | <ul style="list-style-type: none"> <li>• Check and correct the compressed air connection.</li> <li>• Fully open the ball valve.</li> <li>• Straighten the connecting hose.</li> <li>• Check the compressed air system and in case switch the compressor on.</li> </ul> |
| <b>Renfert deflasking chisel fails to start but compressed air can be heard.</b> | <ul style="list-style-type: none"> <li>• Insufficient connecting pressure.</li> <li>• Motor gummed up or blocked.</li> <li>• Damaged piston.</li> </ul>  | <ul style="list-style-type: none"> <li>• Increase the connecting pressure to 4 - 6 bar.</li> <li>• The unit will need to be repaired by our Service Department.</li> <li>• The unit will need to be repaired by our Service Department.</li> </ul>                     |
| <b>Renfert deflasking chisel produces insufficient power.</b>                    | <ul style="list-style-type: none"> <li>• Insufficient connecting pressure.</li> <li>• Ball valve not fully open.</li> <li>• Filter in the connector fitting blocked.</li> </ul>  | <ul style="list-style-type: none"> <li>• Increase the connecting pressure to 4 - 6 bar.</li> <li>• Fully open the ball valve.</li> <li>• Clean the filter.</li> </ul>  |

We reserve the right to make technical changes.

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Renfert GmbH • Untere Gießwiesen 2 • 78247 Hilzingen/Germany