DIAMANT

Descripción

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DIAMANT

Description

Extra hard synthetic die-stone. Type IV, high quality ISO 6873

Uses

Ideal for all types of fixed prostheses, stumps and master models.

Advantages

This product satisfies the most high requirements of the professionals.

Great dimensional stability

Hard and even surfaces

Great resistance to scratches and abrasion .

Good definition of details.

High resistance to breaking.

Tyxotropic gypsum with good consistence for an easy handling and casting

Physical characteristics

Working time:	8 min.
Initial setting:	10 min. (Vicat test according to ISO 6873)
Final setting :	15 min. (Vicat test according to ISO 6873)
Setting time:	28 min.
Setting expansion (2h):	0.08 %
Hardness (24h) :	268 N / mm ²
Compressive strength (1 h) :	60 N / mm ²

Working instructions

Powder/water ratio:	100 g / 20 ml
Working temperature:	20-23°C
Mixing:	mechanical under vacuum: 30 sec. manual: 60 sec.

Add the powder to the demineralised water following the recommended ratio. The initial consistency of the slurry must be thick. In this moment don't add more water above the recommended. While the mixing time is complete, the product acquires the suitable fluidity and stability. When this mixing time has finished, the mix must have a soft and high tyxotropic consistency, flowing easily over the model with a low vibration. After 4 minutes, stop the vibration to guarantee a good crystallisation of the product.

Cautions

1.- USE CAUTIONS:

Use always-clean instruments

Do not mix the product in a container where investments have been mixed.

Use always demineralised water to do the mix. The water from the public mains can affect the correct setting process of the gypsum, making it faster or slower

2.- SAFETY CAUTIONS:

Do not inhale the powder. It's recommended the use of a protective mask.

Keep away from children.

Handle the product in a well-ventilated place.

The handling of this product has to be done only by qualified people

In the event of ingestion, consult a doctor.

For further information, see the safety data sheet for the product.

Storage and expiry

The stability of the product is 18-36 months depending on the package and respecting the storage conditions. (Note: The expiry date is considered with the product in his original packing and before been opened).

Storage: Keep the packages tightly closed in a dry place away from moisture.

After each use the package must be closed to prevent the moisture absorption.

Supply

golden

1101-120 Eco-Pack: 12 Kg (6 x 2 Kg)

1101-M10 Test pack: 100 g

Additional information

The instructions that appear on this sheet are based on our current knowledge and are given only as information without that implies a guarantee from Protechno.

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